

PEDITURN / ONWARD TURNTABLE

The Grimstead Range of Transfer & Repositioning Aids

Pediturn



The Pediturn is a low friction fabric turning disc for indoor use during a standing transfer. The outer surface is of hard wearing anti-slip material, with low friction nylon inner surfaces.

It allows the weight bearing client to pivot easily while in a standing position.

Available in 40cm/15 3/4" diameter.

Onward Turntable

The onward turntable is a turning disc with non slip injection moulded rubber pads and a durable injection moulded plastic body with brass and nylon bushings.

The turntable is ideal for institutional use and is virtually maintenance free.

Available in 33.5cm/13 1/4" diameter.



Difficulty Pivoting – Pediturn and Onward Turntable Solutions

When a client can no longer pivot while standing, the caregivers often find themselves struggling to turn the client to place them onto a seat. This causes them to twist their back causing strain.

The Pediturn or Onward Turntable can be used as an assisted or unassisted turning aid depending on the client's abilities.

General Guidelines



1. Place the Pediturn / Turntable on the floor in a suitable position between the two transfer surfaces.



2. The client's feet are to be placed on the Pediturn / Turntable.



- 3.** Assist the client to a standing position (a transfer belt may be used).



- 4.** Gently turn the client so they are in a position to sit comfortably.



- 5.** Assist them to a sitting position. Remove the Pediturn / Turntable.

Important

The Pediturn should only be used with the red “This Side Up” label up. This ensures maximum strength from the nylon fixing.

In an assisted transfer, assess the client’s suitability for this product (must be able to bear weight).

The two halves of the Pediturn should not be separated. This weakens the nylon fixing and correct operation of the Pediturn cannot be guaranteed.

The Pediturn and Onward Turntable should never be left on the floor as they present a slip hazard.

Cleaning Instructions

The inner and outer surfaces of the Pediturn may be brushed or wiped clean with soapy water or a disinfectant cloth. Do not wash this product or immerse in water. If it should get wet, allow it to drip dry only.

The Onward Turntable can be wiped with soapy water or any disinfectant. The onward turntable can literally be dipped into disinfectant if needed.

Waverley Glen General Guidance Notes

The choice of which equipment to use in any area of work is directly governed by the outcome of the risk assessment process. The interpretation of the information gathered from this process formulates the safe system of work that is implemented. This may include the use of equipment if appropriate. Continuous assessment of clients is important.

It is not possible for suppliers to be prescriptive in the use of equipment, when that equipment may be used in many different clinical situations. The supplier will not know the area in which it is to be used, or the level of care of the patient/client. General guidance is given overleaf, although this will have to be modified to fit the needs of the staff, the patient/client and the environment.

Warranty

Waverley Glen warrants that all new equipment purchased will be free from defects in material and workmanship, under normal use and service, for a period of one (1) year from date of purchase by the original purchaser. Normal wear and tear, damage by natural forces, user neglect and purposeful destruction are not covered by this warranty. Service must be performed by the manufacturer or authorized service depot. Goods returned must be shipped with transportation and/or broker charges prepaid. Manufacturer’s obligation is limited to replacement of parts which have been so returned and are disclosed by the manufacturer to be defective. In no event shall Waverley Glen be liable for any consequential or special damages. Any misuse of or tampering with the equipment shall void this warranty.